

#whdentalwerk



video.wh.com

Now at  
your dealer  
or [wh.com](http://wh.com)



# Class B, simply profitable



lina

# The essentials of sterilization

## Simple and expandable.


Lina is one of the range of W&H sterilizers which offers maximum safety and an excellent price-performance ratio with its Class B cycles and a built-in USB data log.



### Your advantages at a glance

- › The ECO B cycle reduces the cycle time by half
- › Low-noise operation with the patented water separation system
- › User-friendly keypad for simple navigation to all sterilization options
- › Built-in dust filter to protect internal components
- › Programmable cycle start
- › Excellent value for money
- › Integrated automatic water filling valve (optional)
- › Automatic digital cycle recording on USB stick (optional)

lina

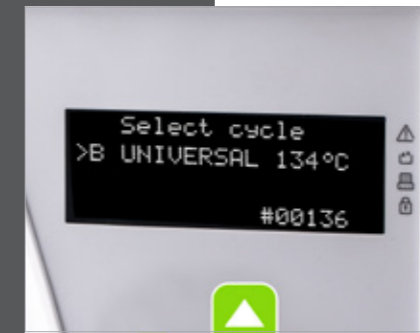

### Class B cycles exclusively

The Lina sterilizer works exclusively with Class B cycles and with short ECO B cycles for smaller loads.



### Simple navigation

All sterilization options are easily accessible with the easily operated keypad.



### ECO B sterilization program

The cycle time can be reduced by half for small loads.



### Built-in data log

The built-in data log means that sterilization cycles can be saved to a USB stick without needing cables and external boxes.



### Barcode label printer

The LisaSafe label printer can be connected to the sterilizer to print barcodes. Combined with the built-in data log your traceability system is enhanced.

# Technical data

	B-Universal 134			B-Prion 134			B-Universal 121
Temperature	134 °C			134 °C			121 °C
Holding time	3.5'			18'			15'
Total cycle duration including drying		ECO-B	Full Load	Typical Load	ECO-B	Full Load	Typical Load
	Lina 17	29'	54'	49'	44'	69'	64'
	Lina 22	33'	64'	55'	48'	79'	70'
Max load: ECO/full/typical loads/ Max porous load				Lina 17: 0.5kg / 4kg / 2kg / 1.5kg Lina 22: 0.5kg / 5kg / 2kg / 1.8kg			
Test cycles	Helix/Bowie Dick/Vacuum test						



	Lina 17	Lina 22
Series:	Lina	Lina
Type:	PRO13-003-17	PRO13-003-22
Chamber size:	17 l	22 l
Power supply:	200 – 240 Vac 50 / 60 Hz; 10 A	
Power consumption	2 kW	
Dimensions overall (W x H x D):	450 x 435 x 599 mm	
Weight:	39 kg	40 kg
Main/Used water tanks:	4 / 3.5 litres	
Working range:	5 to 10 cycles	
Usable space in chamber (W x H x D):	195 x 195 x 297 mm	195 x 195 x 390 mm
Connection type:	1 serial port, 1 USB + 1 serial (optional), integrated automatic water filling valve (optional)	

Lina sterilizers were designed, certified and validated with the most stringent Directives and Standards:

93/42/EEC	Medical Device Directive
2014/68/EU	Pressure Equipment Directive
EN 13060	Small steam sterilizer
IEC 61326-1	Electromagnetic compatibility
IEC 61010-1	Safety requirements
IEC 61010-2-040	Specific requirements for steam sterilizer
› The sterilizer can be validated according to EN ISO 17665-1	

**W&H Sterilization Srl.**  
Via Bolgara, 2  
24060 Brusaporto (BG), Italy  
t +39 035-66 63 000  
f +39 035-50 96 988  
office.sterilization@wh.com  
wh.com

Pictures are for illustrative purposes only. Additional equipment and accessories shown are not included as standard.

20409 AEN Rev. 009 / 16.01.2020  
Subject to alterations