

# **SYRINGE PUMP USER MANUAL**

Please read the manual before using the product;  
Please keep the manual for reference!

## Table of Contents

1. Warnings & Cautions .....	2
2. Introduction.....	5
2.1 Features .....	5
2.2 Application scope.....	5
2.3 Type and specifications .....	5
2.4 Operating conditions .....	5
2.5 Affection on environment and energy .....	5
2.6 The components of each model and definitions .....	6
2.7 Date of manufacture & life span .....	6
2.8 Version of software .....	6
3. Components .....	6
4. Technical and specifications .....	7
5. Installation .....	8
5.1 Installation conditions and technical requirements.....	8
5.2 The Method of Adjusting Pole Clamp .....	9
5.3 Installation method and cautions .....	9
6. External Features .....	9
6.1 Front panel .....	9
6.2 Rear panel.....	12
6.3 Label.....	13
7. Preparation and inspection .....	15
8. Operation Method.....	16
8.1 Operation.....	16
8.2 Alarms and Solutions .....	23
8.3 System Setting and Accuracy Calibration .....	25
8.4 Operation Precautions .....	31
8.5 Contraindications.....	31
9. Malfunctions Analysis and Solutions.....	32
10. Safety Invention and Troubleshooting .....	32
10.1 Safety Invention and precautions.....	32
10.2 Troubleshooting.....	32
11. Maintenance, Inspection, Repair and Recycling.....	33

11.1 Routine maintenance.....	33
11.2 Maintenance during operation .....	33
11.3 Periodic Inspection.....	33
11.4 Normal repair procedures .....	34
11.5 Maintenance for long-time storage.....	34
11.6 Recycling .....	34
12. Transport and storage.....	34
12.1 Precautions during transport .....	34
12.2 Storage conditions.....	34
13. Package list .....	35
14. Open-package Inspection.....	35
15. After Sales Service .....	35
Annex .....	36
Table 1 Classification of alarms and color of alarm indicator light .....	36
Table 2 Alarm conditions and alarm signal delay .....	36
Table 3 Characteristic parameters of alarm signals.....	37
Table 4 Occlusion response characteristic .....	37
Table 5 Starting Curves .....	38
Table 6 Trumpet Curves.....	39

## 1. Warnings & Cautions

Warning: Failure to follow precautions below may result in the risk of death or injury to patients.

- a) The Syringe Pump uses motor-driven screw for medical fluid infusion, but cannot detect leakage caused by disconnection or crack of infusion set. It is required to inspect the infusion status regularly to prevent above problems.
- b) During infusion process, please regularly check the status of the residual liquid inside the disposable sterilized syringe (Hereinafter referred to as the syringes) to ensure correct performance of the infusion.
- c) The Syringe Pump has occlusion detection function. It gives occlusion alarm when the infusion needle fails to insert into intravenous vein properly or the needle deviates from its position inside the vein during infusion. As occlusion alarm is given only after the occlusion pressure reaching a certain value, the area around the needle may already become swollen or bleeding at this time. In addition, the occlusion alarm is not given maybe because the actual occlusion pressure not large enough to reach the occlusion alarm gate, therefore, it needs to check the insertion area regularly. If the insertion area seems abnormal, please take proper treatments such as re-inserting the needle.
- d) Infusion flow blockage that caused by infusion set knotting, filter or needle blocking, or needle occurring thrombosis etc. may lead to pressure increase inside the infusion set. Solving such blockage may be followed by temporary large-volume infusion. The correct method is to clamp the IV set near the insertion area tight before releasing the pressure. Then release the IV set, get rid of the occlusion problem and restart operation. If infusion restarting with blockage remains, occlusion alarm shall sound again and the pressure inside the tube may keep increasing, which may result in disconnection or crack of the tube and further bring harm to the patient.
- e) Use the disposable sterilized syringe consistent with GB15810-2001. When choosing an infusion line, it is advisable to use the syringe with screw and extension tube. Otherwise, it may do harm to patients when the IV tubing is stretched.
- f) The user must install the syringe correctly. Otherwise, infusion may not reach expected performance.
- g) Avoid repeated use or re-sterilizing of disposable syringe. After using, the syringes should be handled in accordance with the appropriate guidelines.
- h) Fix the Syringe Pump well to infusion stand and also ensure the stability of the stand. Be cautious when moving the stand and the Syringe Pump to prevent the Syringe Pump falling off or the stand

collision with surrounding objects.

- i) The Syringe Pump cannot use with possible large negative or positive pressure piping such as extracorporeal circuit. As in such case, the Syringe Pump cannot ensure infusion accuracy and correct alarm functions.
- j) The Syringe Pump cannot use for blood transfusion.
- k) Do not use the Syringe Pump near inflammable liquid or gas.
- l) Do not store or use the Syringe Pump in humid environment or environment with chemically active gases (including gas for sterilization). Such environments may have an impact on internal electronic parts and thus bring degradation or damage to their functions.
- m) The syringe pump cannot be powered by the vehicle-mounted power supply. If the vehicle-mounted power supply is used, please add the voltage stabilizer or the power inverter complying with the safety regulations so as to let power supply output become the stable voltage complying with the input power of the syringe pump; if not, the syringe pump may be damaged seriously.
- n) At any time when conducting the target-controlled infusion, please provide the all-around monitoring for the patient; user must be only the experienced and trained anesthetist, and he/she must know about the respiratory tract and breathing of the patient, and has been trained in sense of the cardio-pulmonary resuscitation skill.
- o) Please use to meet the relevant laws and regulations, with a valid medical device registration certificate of the infusion tube, or cannot guarantee the accuracy of infusion and normal detection alarm.

**Cautions:** Failure to follow cautions below may lead to injury of operator/patient or loss of property.

- a) Inspect the Syringe Pump before use, making sure it can work normally. If any malfunction is found, stop operation immediately and contact the distributor or the manufacturer. Besides, adhesion or leakage of medical liquid may lead to malfunction of the syringe Pump. Therefore please clean the Syringe Pump and store it properly after each use.
- b) When use the Syringe Pump the first time after purchasing or after long-time of storage, please connect it to AC power source and charge it for at least 10 hours with power on, or 3 hours with power-off. If not fully recharged, the internal battery can't support the Syringe Pump with enough power in case of AC power failure.
- c) If using near electric cautery equipment, the Syringe Pump may result in wrong operation due to the

high frequency wave of electric cautery equipment. If the Syringe Pump has to be used with electric cautery equipment, please take proper measures as follows:

- (1) Avoid using the Syringe Pump along with old-fashioned electric cautery apparatus (open vacuum tube).
  - (2) The distance between Syringe Pump and the body of electric cautery apparatus or its power source should be more than 25cm.
  - (3) The Syringe Pump shall not use the same electric cabinet as that of electric cautery apparatus, and having reliable ground connection.
- d) Do not use mobile phone, wireless device or cardiac defibrillator within 1 meter near the Syringe Pump. Otherwise the high frequency noise/signal may cause wrong performance of the Syringe Pump. Make sure the Syringe Pump has ground connection and do not use the same power socket with that for the above-mentioned devices.
- e) The Syringe Pump cannot use in area with radiotherapy equipment or magnetic resonance (MR) equipment or hyperbaric oxygen therapy.
- f) Do not use pointed object like pen-tip or finger nail etc) to press on keys of the Syringe Pump. Otherwise, the keys or the mask may suffer premature damage.
- g) Keep the infusion set and the Syringe Pump a certain distance from the AC power source and DC socket to prevent the medical liquid from splashing or dropping onto the socket to incur shortage of circuit. In addition, make sure the power plug and socket are dry before connecting to power source.
- h) Try to use the medical liquid when it reaches or near room temperature.
- i) In normal conditions, try to use AC or DC power source to extend battery service life. When use AC power source, making sure it is well connected to ground and please use the power cord that is standard configuration with the Syringe Pump. Just use battery when there is difficulty in ground connection or without AC power (such as AC power failure or mobile infusion).
- j) Pay more attention to occlusion when infusion at low rate. The lower the rate, the more time needed for detecting occlusion, thus there may be a long interval of infusion interruption.
- k) When using computer port, it may suffer interference from devices such as electric cautery apparatus, mobile phone, wireless device or cardiac defibrillator etc. Please try to keep away from the above-mentioned devices.
- l) If the Syringe Pump falling off or suffering collision, stop using it immediately and contact the distributor or the manufacturer. Even there is no damage on appearance or no malfunction alarm, the internal

parts may have damaged.

- m) The Infusion Pump must be operated by well-trained professionals such as doctor, nurse and medical device expert.
- n) Do not disassembly or modify the Syringe Pump or use it for other purposes other than normal infusion. Otherwise, the manufacturer takes no responsibility.

## **2. Introduction**

### **2.1 Features**

User-friendly interface, easy operation.

2.8 inch colorful LCD with detailed menu.

Internal multiple reliable design and alarm functions, more stable and safer infusion.

Arc shape and easy cleaning.

### **2.2 Application scope**

It is used in hospitals where patient need intravenous infusion.

### **2.3 Type and specifications**

This product belongs to class I , type CF. It is on continuous operation and with internal battery. It cannot be carried by patient for mobile use. It can't be used in mixed gases of flammable anesthetic gas with air, or of oxygen or nitrous oxide with flammable anesthetic.

### **2.4 Operating conditions**


a) Temperature: 5°C-40°C

b) Relative humidity: 10-95% (no frosting)

### **2.5 Affection on environment and energy**

This product may have certain electromagnetic radiation which may influence other devices. In such case, please take proper measures to reduce the interference such as re-locating the Syringe Pump, or using AC power from a different source etc.

## 2.6 The components of each model and definitions

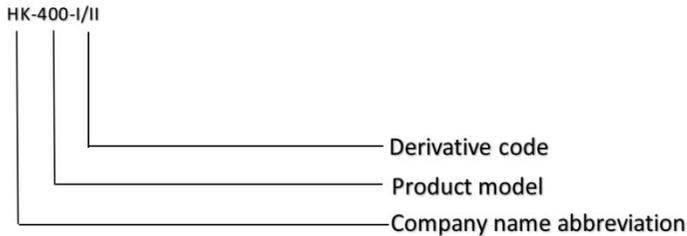

## 2.7 Date of manufacture & life span

The life span of the infusion pump (battery is not included) and its cable is 5 years. Please refer to label for date of manufacture.

## 2.8 Version of software

The version of the user manual for infusion pump's software is V01.

## 3. Components

The Syringe Pump is mainly composed of 5 parts: microcomputer system, pump body, detection device, alarm system and input & display part.

**Microcomputer system:** the brain of the whole system, giving an intelligent control and management to the whole system and processing signals detected, adopting double CPU.

**Pump body:** the heart of the whole system and the driving force of transfusing medical liquid. It uses step motor driving screw to push the syringe plunger forward.

**Detection device:** mainly containing sensors, such as ultrasonic sensor (for detecting motor running and reversing) and pressure sensor (for detecting occlusion) etc. They can detect corresponding signals, which after being amplified and transferred to microcomputer system for signal processing and thus incur control instruction for corresponding operation.

**Alarm system:** The signals detected by the sensor, after being processed by the microcomputer,

shall incur alarm control signal and then at the response of alarm system, which alert the user for immediate correct operation. It contains mainly photoelectric alarm (light emitting diode) and audible alarm (loudspeaker and buzzer) etc.

Input & display part: Press keypad to set all parameters such as infusion volume and flow rate. LCD displays all parameters and present operation status.

#### 4. Technical and specifications

Technical and parameters	
Applicable syringe	5, 10, 20, 30, 50(60)ml disposable sterile syringes
Volume to be infused (VTBI)	(0-1000)ml
KVO rate	(0.1-2)ml/h, preset by the user; default: 0.1ml/h
infusion rate	5ml syringe: (0.1-150)ml/h 10ml syringe: (0.1-300)ml/h 20ml syringe: (0.1-600)ml/h 30ml syringe: (0.1-900)ml/h 50ml(60ml) syringe: (0.1-1500)ml/h Infusion accuracy: $\pm 2\%$
Bolus rate	5ml syringe: (100-150)ml/h 10ml syringe: (100-300)ml/h 20ml syringe: (100-600)ml/h 30ml syringe: (100-900)ml/h 50ml(60ml) syringe: (100-1500)ml/h
Purge	5mlsyringe: 150 ml/h 10mlsyringe: 300 ml/h 20mlsyringe: 600 ml/h 30mlsyringe: 900 ml/h 50(60ml)syringe: 1500 ml/h

Occlusion pressure	(40.0-160.0)kpa;3 levels (adjustable): low, middle, high; default: middle
Water Proof Level	IP24
AC power	100-240V 50/60Hz
Battery	Li_Polymer 7.4V 1900mAh; Recharge time: 10h with power on, 3h with power off. Running time: more than 6h at rate of 5ml/h, environment temperature 25°C after being fully charged.
Power consumption	35VA
DC	DC 12V ±1.2V
Fuse	Slow fuse Specification:250V 2A
Operating conditions	Environment temperature 5°C ~40°C Relative humidity: 10-95% (no frosting) Air pressure: 86kPa~106kPa
Dimensions	300(L)x 130(H)x 125(W, not including pole clamp)mm
Net weight	1.8kg
<b>Glossary</b>	
KVO	Keep vein open
Bolus	The amount of fast infusion
Purge	Rinse
Anti bolus	Diminishes the volume of unwanted Bolus after removal of the occlusion.
Infusion tubing	Syringe and extension tube

## 5. Installation


### 5.1 Installation conditions and technical requirements

The Syringe Pump can be fixed to a vertical IV pole or horizontal bar with diameter of 12-35mm, or on platform with slope angle not exceeding 5°.

## 5.2 The Method of Adjusting Pole Clamp

If the pole clamp is not in the same direction with that of IV stand or bar, adjust it to suit the direction of the IV stand or bar.

Loose the clamp screw, rotate the pole clamp 90° clockwise or counterclockwise, and tighten the fixing clamp screw, screw the clamp knob to fix the pump.

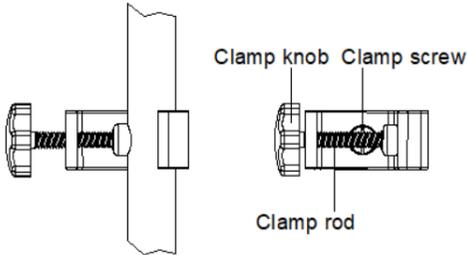

## 5.3 Installation method and cautions



Method 1: Put the syringe pump on stable platform.

Method 2: Fix the syringe pump to IV Pole as per below steps:

- (1) Rotate the clamp knob screw out the clamp rod, leave space for IV pole.
- (2) Clamp the IV pole, screw down the knob to fix the position of syringe pump. Should hold the syringe pump during the installation; only release the hand after screwed tightly to avoid falling.

## 6. External Features

### 6.1 Front panel

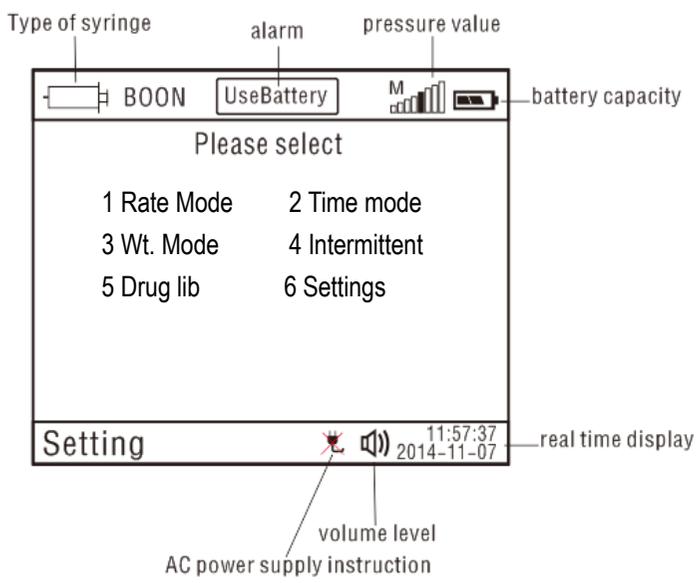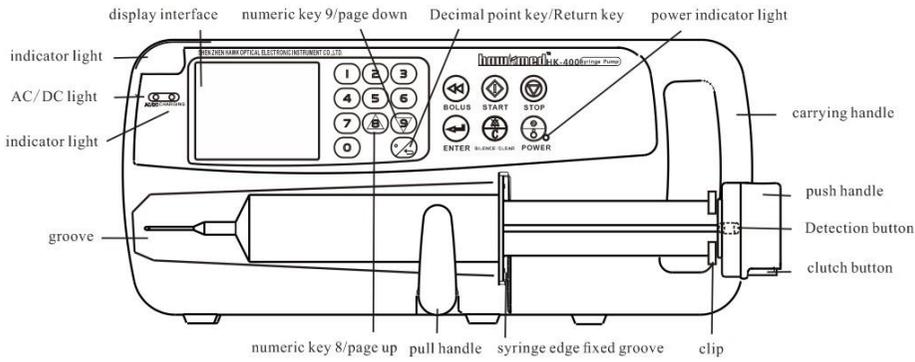


( Diagram1 LCD display )

Description	Functions
BOLUS key	<p>In 'stop' status, press &amp; keep finger on 'bolus' key, the pump starts purging. After releasing the finger, purging stops.</p> <p>During operation, press &amp; keep finger on 'bolus' key, the pump starts bolus infusion (bolus rate preset by the user). Release the finger, bolus infusion stops and the pump continues infusion at original rate.</p>
START key	In 'stop' status, press this key to start infusion.
STOP key	Press this key to stop infusion.
SILENCE /CLEAR key	<p>1.Press this key to silence the alarm signal</p> <p>2 clear value when inputting parameters</p>
POWER key	<p>Switch on / off the Syringe Pump.</p> <p>1.In 'power off' status, press this key until LCD screen displays, which means the pump is switched on.</p> <p>2.In 'stop' status, or in 'alarm' case, press this key and the pump shall be switched off.</p>
ENTER key	Make the parameters adjustable or save the parameter newly setting
AC / DC indicator light	If on, it indicates there's AC/DC input; if off, it indicates there's no AC/DC input.
Indicator light	<p>Indicator light indicates operating status/alarms cases. The green indicator light flashes when the infusion is in normal progress.</p> <p>If high-priority alarm occurs, the indicator light shall turn red and flash.</p> <p>If middle-priority alarm occurs during operation, the indicator light shall turn yellow and flash.</p> <p>If low-priority alarm occurs during operation, the indicator light shall turn yellow but not flash.</p> <p>* Please refer to Annex Table I for priority of alarm classification.</p>
Charging indicator light	<p>This indicator light on means the battery is recharging.</p> <p>This indicator light off means the battery is not charging.</p>

numeric key 8/ page up	<ol style="list-style-type: none"> <li>1. In the numerical input status, it is digital key to enter the value of 8</li> <li>2. In the menu selecting status, press this button to turn the page (upturning).</li> </ol>
numeric key 9/ page down	<ol style="list-style-type: none"> <li>1. In the numerical input status, it is digital key to enter the value of 9</li> <li>2. In the menu selecting status, press this button to turn the page (Page Down).</li> </ol>
Decimal point key/ <b>Return key</b>	<ol style="list-style-type: none"> <li>1. Decimal point key works in the numerical input status.</li> <li>2. In the menu selecting status, press this button to return to the last operation interface.</li> </ol>
Display interface	Display Settings/ parameters/ working status etc.
groove	Syringe installation location
Pull handle	Used for fixation the syringe and avoid it disengage.
Syringe edge fixed groove	Used for fixation the syringe edge.
Push handle	Press and move clutch button on push handle, which drive the syringe pump screw moving
Detection button	To detect whether the syringe handspike is installed in place.
Clutch button	Keep pressing clutch button and move push handle freely.
clip	Use to fix the syringe handspike and avoid syringe handspike disengage

## 6.2 Rear panel

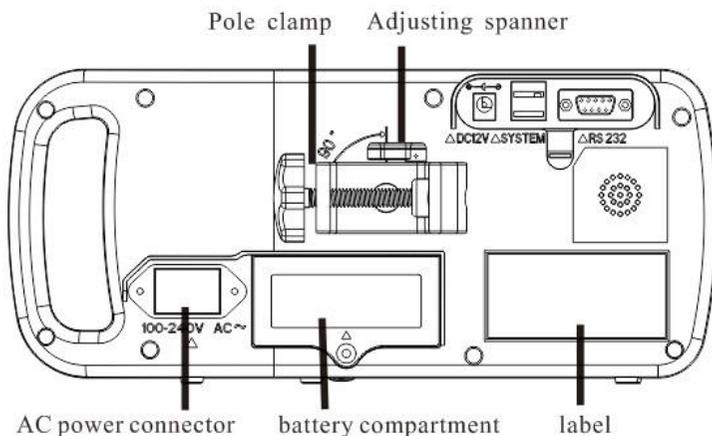

Description	Functions
<b>Pole clamp</b>	It is used to fix the syringe Pump on IV stand. Loosen the clamp screw to change the direction of pole clamp. (Horizontal or vertical optional)
<b>Battery compartment</b>	Battery location. Open it from the bottom of machine.
<b>AC power connector</b>	AC power connector
<b>RS232 port</b>	It is used to connect syringe pump to standard PC to transfer infusion history records. Note: This process must be carried out when machine in non-infusion state. The RS232 communication line must use shielded wire.
<b>DC12V input</b>	It can be connected to DC power supply (12V±1.2V).
<b>SYSTEM interface</b>	Used for joint control with the other devices of our company.

## 6.3 Label

### 6.3.1 Product label (on the back shell)



The label contains information such as manufacturer, date of production, product serial No.,

classification, waterproof level, etc.

### 6.3.2 Symbols and significance

(Table 1)

Symbols	Descriptions
	Production batch No.
	Product serial No.
	Caution, consult accompanying documents
	Consult instruction for use
	Type CF
	Protective Earthing
IP24	Waterproof level: dripping water by slope angel 60°
	AC power
	DC power
	Dispose in environmental-friendly way
	Date of production
	manufacturer

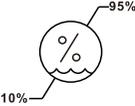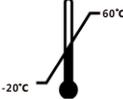	<p>Caution Against Wet</p>
	<p>Fragile. Handle with care!</p>
	<p>Keep upright during transport</p>
	<p>5 layers at most of the same package</p>
	<p>Transport package humidity 10~95%</p>
	<p>Transport package temperature -20°C~60°C</p>

## 7. Preparation and inspection

Whether the Syringe Pump is a new one, or it has been stored for a period of time, or it just has been repaired, please check the following terms before use:

- (1) The outlook remains good, clean, no crack and no leakage.
- (2) All keys are responsive, no invalid key or stuck key.
- (3) Syringe pump push handle could move freely.
- (4) The power cord can be plugged in tight, not easy to loose.

(5) If Syringe Pump worked on internal battery only, charge it fully before use and also make sure the battery is still valid for use.

## 8. Operation Method

### 8.1 Operation

The whole infusion operation contains the following processes:

- 1) Fix the Syringe Pump and connect it to AC/DC power.
- 2) Switch on / off
- 3) Install the Syringe
- 4) Set infusion parameters
- 5) Purge the air in line
- 6) Start infusion
- 7) Bolus infusion
- 8) Stop infusion
- 9) infusion completion
- 10) Replace Syringe

#### 8.1.1 Fixed the syringe pump, connect it to AC/DC line

Adjust the pole clamp to fix the Syringe Pump properly to a stand/bar/cage and connect it to AC/DC power. At this time, the AC/DC indicator light  (on upper left corner) shall be on.

#### 8.1.2 Switch on/off.

Switch on: Press Power key  few seconds, the pump will be switched on and do self-test, and shows Self-testing on the display, it will test: Communication Info, Press sensor, Potentiometer, Dedicated IV set, AC Info and Battery Info. The test result (OK or FAIL) will be displayed on LCD.

Pump self-test information after switching on:

- (1) Communication information: OK indicates CPU normal; FAILED indicates CPU abnormal;
- (2) Battery information: OK indicates internal battery normal; FAILED indicates internal battery abnormal;
- (3) AC information: OK indicates AC connected well; FAILED indicates AC fail;
- (4) Pressure sensor: OK indicates normal pressure detection; FAILED indicates: abnormal pressure detection;
- (5) Potentiometer: OK indicates normal Potentiometer; FAILED indicates: abnormal Potentiometer;
- (6) Dedicated IV set: OK indicates normal dedicated IV sensor; FAILED indicates: abnormal

installation of dedicated IV set or IV set is installed in the pump already;

**Attention: 1. If the potentiometer is abnormal, the system will ask “Is the syringe installed?”, please choose the answer according to the facts, the system will predicate per your choice.**

**2.Please keep an eye on the self-test, if any abnormal, please contact our company, DO NOT force to use it.**

Press POWER key  for about 2 seconds to turn off the machine.

### 8.1.3 Install the syringe

(1) Pull the pull handle of syringe pump to the end, then turn left 90° and fix it.

(2) Full the syringe with medical liquid, connect it to extension tube and scalp needle, purge the air in line, then install the syringe in groove and syringe edge fixed groove.

(3) Turn the pull handle right 90°. It will rebound and compress the syringe.


(4) Press the clutch button on push handle tightly; move the push handle to the end of syringe handspike, release the clutch button, the clip will grip the end of syringe handspike automatically. Meanwhile, the size of syringe will be displayed on LCD.

**Attention: (1) Make sure there's no air bubble in the syringe.**

**(2) Please make sure the end of syringe handspike is fixed on push handle groove, otherwise they will be no liquid output.**

### 8.1.4 Set infusion parameters

#### 8.1.4.1. Infusion Mode Parameter Table

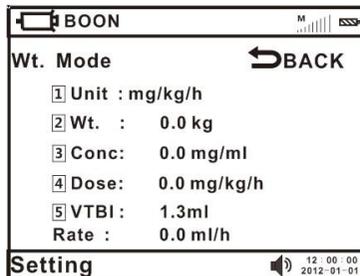

Wt. Mode	
1] Unit :	mg/kg/h
2] Wt. :	0.0 kg
3] Conc :	0.0 mg/ml
4] Dose :	0.0 mg/kg/h
5] VTBI :	1.3ml
Rate :	0.0 ml/h

Setting  12:00:00  
2012-01-01

(1)As above diagram show, e.g. Setting “Unit”, its corresponding numerical value is 1, so press

numeric key “1”. “Unit” displays in a white text and that means it can be editing. When  displays on

the right, press numeric key "8" or "9" to set the parameters. Press ENTER key to save the value newly setting, then the white text disappears.

(2) If setting weight, input the number directly after the white text appearing, press ENTER key to save it. If input wrong number, press CLEAR key to reset.


**Attention:**

(1) It can only continue to set other parameters after pressing ENTER key to save the value newly setting.

(2) If the setting value out of system default limit, the system will only display the minimum or maximum default value. Take body weight for example, if system default value is 3-150 kg, input 2, it will only display 3; input 160, it will only display 150.

8.1.4.2 Submenu (Infusion Mode) Selection

**(1) Rate mode**

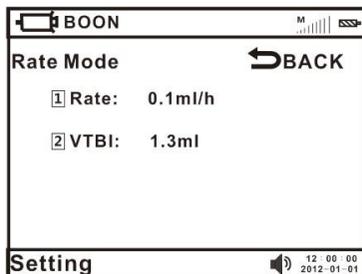


On syringe pump main menu, press corresponding number key of Rate mode enter the parameter setting interface. Flow rate and volume to be infused (VTBI) can be setting.

**Flow rate:** Press numeric key "1", input the flow rate in white text. If input wrong value, press CLEAR key to remove and reset. Press ENTER to save value and exit.

Setting VTBI parameter as per above methods.

Return: Press RETURN key  back to main menu (Diagram3) if want to reset the infusion mode.

**(2) Time mode**

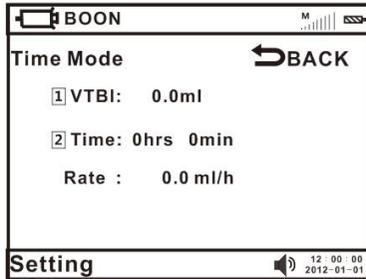

On syringe pump main menu, press corresponding number key of time mode entering the parameter setting interface. VTBI and time can be setting.


**VTBI:** Press numeric key “1”, input the VTBI (volume limit) in white text. If input wrong value, press CLEAR key to remove and reset. Press ENTER to save value and exit.

Setting **Time** parameter as per above methods. After setting all parameters, the flow rate will be calculated automatically.

**Attention: When the setting flow rate is more than the maximum limit, it will remind “Rate: >1500ml/h”. Then please reset as above steps.**

Return: Press RETURN key  back to main menu (Diagram3) if want to reset the infusion mode.

### (3)Body Weight Mode

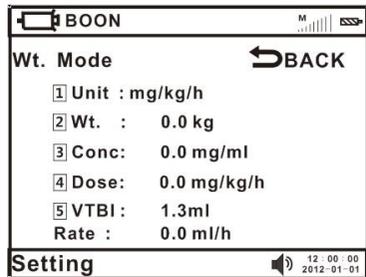

On syringe pump main menu, press corresponding number key of Body weight mode entering parameter setting interface. Unit, Body weight, concentration, dose, VTBI can be adjustable. The flow rate will be calculated automatically after setting the other parameters.

**Unit:** Press numeric key “1”, there are 4 optional units “mg/kg/h、mg/kg/min、ug/kg/h、ug/kg/min”. Press numeric key “8” or “9” to choose the required unit, press Enter key to save value and exit.

**Body weight:** press numeric key "2", input the weight in the white text. If input wrong value, press CLEAR key to remove and reset. Press ENTER to save value and exit.

Set the other parameters in the same way.


Return: Press return key  back to main menu if want to reset the infusion mode.

**Attention:** The flow rate will be changed when switch the Unit.

1. If there is parameter forgotten to set, it will remind "Parameters haven't been set" when start infusion. Press Return key  back to parameter setting interface, check and set it.

2. When the setting flow rate is larger than the maximum limit of syringe pump, it will remind "Rate out of range" when start infusion. Press Return key  back to parameter setting interface, check and set it.




#### (4)Intermittent Mode

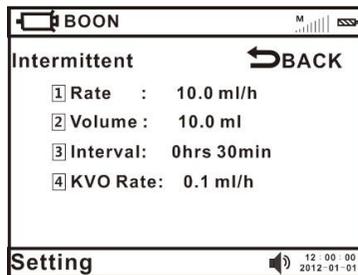

On syringe pump main menu, press corresponding number key of Intermittent mode entering parameter setting interface. Flow rate, volume limit each time, interval time and KVO rate can be setting. Press Enter key to save value and exit.

Return: Press RETURN key  back to main menu if want to reset the infusion mode.

#### (5)Drug lib

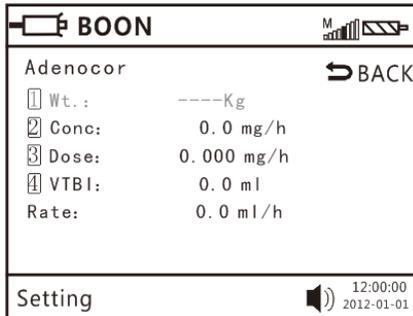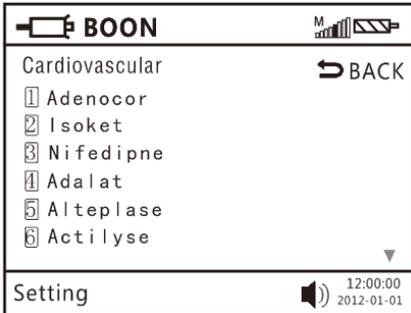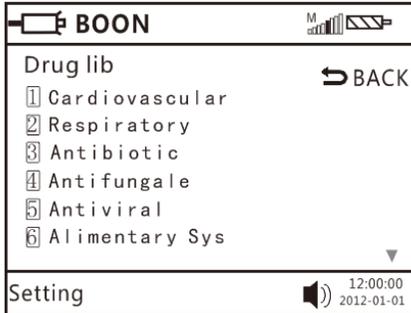


press the corresponding number key to enter the parameter setting interface, according to the demand to select the drug category, enter the drug name menu, set the parameters correct, press the Enter key to save.

### 8.1.5 Purge

In 'stop' status, press & hold on BOLUS key until all air inside the tube is purged.

### 8.1.6 Start Infusion

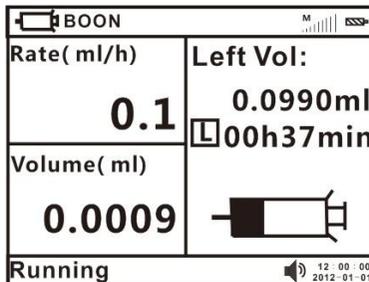

(Rate mode running interface)

Press START key after setting all the parameters correctly. During infusion, only purge key and stop key shall function. And it will show "operating" on the left bottom.

### **8.1.7 Bolus Infusion**

There are two methods to purge. They are manual bolus and auto bolus. Please follow by the following procedures:

- 1) Manual bolus: During infusion process, press bolus key and hold on. The pump will purge at preset bolus rate. It will return to previous infusion rate after releasing your finger.
- 2) Auto bolus: After setting bolus rate, it will purge at bolus rate by single clicking bolus key.

**Attention: Different syringes have different bolus rate. Bolus rate can be setting on the system menu, please refer to 8.3.4 in user manual.**

### **8.1.8 Stop Infusion**

During infusion, press STOP key to stop infusion. It will show the  $\Sigma$  (volume infused) and other adjustable parameters, set them as required, press START key to work again. Choose "End injection" back to setting menu; the  $\Sigma$  (volume infused) will clear to 0 automatically.

### **8.1.9 Infusion completion**

After VTBI completion or  $\Sigma$  (volume infused) reaching 9999.9ml, the pump shall start KVO Function automatically and give over alarm. Press STOP key to stop infusion.

★ KVO Function means keep patient's vein open by keep infusion at a pre-set low rate.

### **8.1.10 Replace Syringe**

(1) Pull the pull handle to the end then turn left 90°; keep pressing clutch button and take out the syringe.

(2) Install syringe as per 8.1.3 in user manual.

### **8.1.11 to change infusion rate during infusion**

When the pump is working, press numeric key "5" to enter into the interface of changing infusion rate.

Setting procedure: Infusion rate: Press numeric key "1" to make it editable. Then enter infusion rate value you want to set. If any error, press "SILENCE/CLEAR" key to clear and enter correct one.

Preset: Press numeric key "2" to make it editable. Enter into preset value. If any error, please press

“SILENCE/CLEAR” key to enter again. Save the value after confirmation.

Return to infusion interface: after setting infusion rate and preset volume, press “start” key to return to infusion interface. But please note:

- (1) Infusion rate should be more than “0”.
- (2) Preset volume should be more than real time total volume.

## 8.2 Alarms and Solutions

During infusion preparation and process, alarms may occur as follows. Please treat them as below instructions.

Table 2(Refer to Annex Table1 for corresponding alarm parameters)

Name of alarms	Cause for alarms	Solutions
Handle off alarm	The end of syringe handspike is not correctly installed into push handle groove / not gripped by clip.	Press SILENCE key to clear the alarm signal. Reinstall the syringe correctly.
syringe off alarm	During operation, take out the syringe or pull handle does not compress on syringe.	Press SILENCE key to clear the alarm signal. Reinstall the syringe correctly.
Empty Alarm	The VTBI is complete .	Press Clear key to clear the alarm.
Occlusion alarm	The infusion line is blocked	Press SILENCE key to clear the alarm signal.
Battery exhaust alarm	Battery icon shows blank when operate on battery	Plug AC power cord to clear the alarm. If AC power cord is not plugged in, the alarm shall cannot clear (will continue to alarm for 3mins at least)
Low Battery alarm	Battery icon shows only 1 grid when operate only on battery	Press SILENCE key to clear the alarm. If AC power cord is not plugged in, the alarm shall not clear (in this case, it can infuse at least 30min at medium flow rate)

Near empty alarm	The syringe will be empty soon.	Press SILENCE key to clear the alarm signal, but will still display on the interface.
Almost Done alarm (infusion near over)	The VTBI is almost completed	Press SILENCE key to clear the alarm signal, but will still display on the interface
Finished alarm	The VTBI is completed	Press SILENCE key to clear the alarm signal.
Use Battery alarm	AC power cord is not plugged in.	Press SILENCE key or connect to AC power to clear alarm signal.
AC fail alarm	Power failure or AC power plug off after switch on.	Press SILENCE key or connect to AC power to clear alarm signal
No Operate alarm	If there is no operation on machine for 2 minutes after switch on, it shall give 'no operate' alarm.	Press any key to clear the alarm
0xE0, 0xE1 0xE2, 0xE3	1. 0xE0: data communication error.	Reboot the machine and load the parameters of last infusion to try operation again. If problem still occurs, contact distributor /manufacturer for repair
	2. 0xE1: The syringe Pump's driving system has problem	Reboot the machine and load the parameters of last infusion to try operation again. If problem still occurs, contact distributor /manufacturer for repair
	3. 0xE2: The Infusion Pump's motor has problem	Reboot the machine and load the parameters of last infusion to try operation again. If problem still occurs, contact distributor /manufacturer for repair

	<p>4. 0xE3: The Infusion Pump's data storage has problem</p>	<p>Reboot the machine and load the parameters of last infusion to try operation again. If problem still occurs, contact distributor /manufacturer for repair</p> <p><b>★After restoring factory default setting, you need to calibrate the syringe parameters again.</b></p>
--	--	--

### 8.3 System Setting and Accuracy Calibration

On main menu, press numeric key entering system setting.

#### 8.3.1 Select Syringe brand

Press numeric key “1”, when it become white text, press numeric key “8” or “9” to choose the syringe brand. Press Enter key to save and exist.

**Attention:** The syringe pump only stores a few brands of syringe. When brand of syringe being used is not found here, please turn to instruction 8.3.2.

#### 8.3.2 Self-Defined Syringe

The system is built in many brands of syringes: Boon, HuaFu, BRAUN, FL, KDL, HD,SG,LX,JR.

##### (1) Brand name setting procedure.

Select “Settings” to enter into setting interface. Then select “2 Set Syringe Para”. Then select “1 Set the brand name” to set brand name.

Setting methods:

please press one of the numeric keys 1-5 to make it editable.

Press numeric keys 1or2 or3 or 4 or5 or 6 or7 to enter into wanted letters.

Note:1=A,B,C,D

2=E,F,G,H

3=I,J,K,L

4=M,N,O,P

5=Q,R,S,T

6=U,V,W,X

7=Z,Y

please press 8 (up) 9(down) to shift from one to another letter

## **(2)Self calibration procedure:**

Take 50ml syringe for example, pull the plunger(handspike) to 50ml scale and install it correctly.

Enter into settings interface. Press numeric key "2 Set Syringe Para" to enter into self calibration interface. In the "Set the Syringe Para" interface, select "2 Set the Parameters" and then choose one brand name like Brand B. In this brand setting interface, choose corresponding syringe like 50 ml syringe. There are two calibrating Methods.


Method 1: auto calibration

Select "1 Auto measure syringe". A popup menu will show "Pull syringe handspike to 50ml scale, install it to machine". Select "1 Yes" to start calibration. After finish, select "3" to confirm it.

Method 2: manual calibration

Select "2 Manual measure syringe". Then measure the length of scale( from 0-50ml) and the length of tail. Select "1 Length: ml" to enter into scale length and select "2 Tail: ml" to enter into tail length. Then press Key "3"to confirm it. Now the manual calibration is done.

**Attention: please choose correct brand name before using the pump in order to avoid alarm and make it more accurate.**

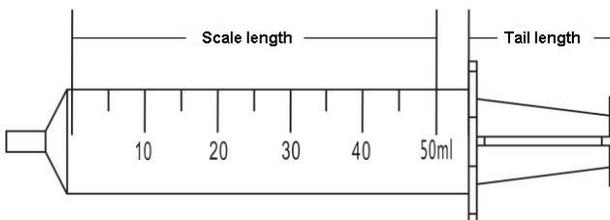

### **8.3.3 Set occlusion Alarm level.**

This parameter should be calibrated by pressure scale. Users should adjust this parameter according to the selected syringe.

Occlusion value is range from 40Kpa to 160Kpa.

Press numeric key “3”, appear white text box, press numeric key “8” or “9” to select “High”/ “Middle”/ “Low” as per real situation. Press Enter key to save value and exit.

If the actual pressure value measured upon Occlusion alarm is higher, adjust occlusion alarm level to a smaller one. Otherwise, adjust occlusion alarm level to a larger one.

After setting, re-measure the actual pressure value to ensure actual pressure value is within occlusion alarm level range.

### **8.3.4 Bolus Rate Setting**

There are two methods for Bolus, manual Bolus and auto Bolus, operation as below:

Manual Bolus: After power on the pump, press STOP+ left 1st soft key to enter Settings, select Bolus, set Bolus rate and VTBI: 0, press ENTER to save the value. During infusing, keep pressing BOLUS key then it will run at Bolus rate; if release the BOLUS key, it will run at original rate, not Bolus rate.

Auto Bolus: After power on the pump, press STOP+ left 1st soft key to enter Settings, select Bolus, set Bolus rate and VTBI >0, press ENTER to save the value. During infusing, press BOLUS and release, it will run at Bolus rate until finishing the VTBI, then run at original rate.

### **8.3.5 KVO Rate Setting**

Press numeric key “5” to set the KVO rate, input the value and press enter key to save.

### **8.3.6 Key Lock setting**

Press numeric key “6” to set key Lock. Press numeric key “8” or “9” to select “Never” “After 5 min” “After 4 min” “After 3 min” “After 2 min” “After 1 min”. Then press ENTER key to save the value and exit.

Remark: Setting as “After 5 min” “After 4 min” “After 3 min” “After 2 min” “After 1 min” means all keys shall be locked (except POWER key) “After 5 min” “After 4 min” “After 3 min” “After 2 min” “After 1 min” if no operation on keys. This icon shall display on LCD.

To unlock the panel, press ENTER key and numeric key “2” together.

**Press numeric key “9” to enter the next page of system setting.**

### **8.3.7 Alarm Volume Setting**

Press numeric key “1” to enter the sound level setting. Press numeric key “8” or “9” to select “High” or “Low”, Press Enter key to save value and exit.

### **8.3.8 LCD backlight level setting**

After entering ‘parameter setting interface’, press numeric key “2”, then press numeric key “9” to select “Bright”, “Dark” or, 5 min, 4 min, 3 min, 2 min, 1 min, press ENTER key to save the value and exit.

Remark: Selecting “5min” “4min” “3min” “2min” “1min”etc. means select the corresponding backlight level. The LCD shall automatically darken in “5min” “4min” “3min” “2min” “1min”etc if no operation on keys.

### **8.3.9 Key Sound Setting**

After entering ‘parameter setting interface’, Press numeric key “3”, select ON or OFF。

### **8.3.10 Set Date and Time**

After entering ‘parameter setting interface’, press numeric key “4” for year/Month/day/hour/minute/second setting. Press ENTER key to save the value and exit.

### **8.3.11 Set pressure base value**

After entering ‘parameter setting interface’, press numeric key “5”, then input pressure value and press ENTER key to save the value . The smaller the pressure base value, the more sensitive the pressure alarm.

### **8.3.12 View the event logs/alarm records**

After entering ‘parameter setting interface’, press numeric key “6” for ‘Log’. Select ‘1 Upload log’, all infusion records can be viewed on computer (only available when connect the pump to computer by RS232 interface). Select ‘2 View log’, the pump can directly display the latest infusion/alarm information. Press “Back” the pump shall return to ‘parameter setting interface’.

(1) Upload log: upload infusion records to computer. Please refer to steps as follows:

a. Connect the syringe pump to a computer with RS232 cable.

Computer (in power-on status)-click "start" (left bottom corner)-click "programs"-click

"accessories"-click "communication"-click "hyper terminal"-click disconnect icon . Then in "file" menu, select "properties" and set COM interface (according to actual 232port).

b. In 115200 properties interface, click "configure" and set "baud rate" as 115200 and data flow control as Xon / Xoff.

c. After setting is complete, click call icon  to connect to terminal.

d. In Hyper Terminal interface to select "Transfer - Capture Text" , recommending set up a txt named after a syringe pump serial number on the computer, and then click "Start".

e. Press numeric key "1" , upload records to computer terminal. Press "transfer - capture text " after finishing uploading. And all infusion/alarms records can be reviewed on the txt that setting previously. After finishing uploading, the syringe pump returns to previous menu automatically.

(2) View Log: Press numeric key "2" to view latest 1500 pieces of infusion records/alarm information. Press 'Prev' to check the previous records or 'Next' for next records. Press 'Back' to return to previous menu.

(3) Return : Press "Back" to return to parameter setting interface

(4) After shutdown, the electronic memory is five years.

### 8.3.13 "No Operate" alarm setting

After entering 'parameter setting interface' press numeric key "1" to select "ON" or " OFF" . "No Operate" alarm setting as on: in 'stop' status, "No Operate" alarm shall sound when no operation on keys in 2 minutes.

#### **8.3.14 Almost Done alarm**

After entering 'parameter setting interface' press numeric key "2" to select ON or OFF. If setting as ON, "Almost Done" alarm shall sound 3 minutes before VTBI is complete.

#### **8.3.15 Night mode on and off setting**

After entering 'parameter setting interface', press numeric key "3" for 'NIGHT' and select ON or OFF.

If setting as ON,  shall display on LCD. Key shall be off; the screen shall turn dark after 1 minute if no operation on keys; the top indicator light shall be off during infusion. (If there is any alarm, the light shall be on.)

#### **8.3.16 Select language**

After entering 'parameter setting interface' Press numeric key "4" to enter language setting interface .Select "Chinese", "English" or "Cancel".

#### **8.3.17 WIFI configuration and WIFI module (optional)**

The pump supports connection to HK-M1000 fluid management system wirelessly, it works through WIFI module (optional) and router (optional). Steps as below:

WIFI configuration: After entering Settings, select WIFI, set it ON, then active the WIFI function. It will connect with the WIFI network which connected successfully last time.

WIFI module configuration: Connect with HK-M1000 system wirelessly, send the infusion data, alarms, reminders etc to HK-M1000 system. For more information please refer to User manual of HK-M1000 fluid management system.

Attention: The configuration must be set by the engineers we admit, or the servicer we appointed.

#### **8.3.18 Restore default**

After entering 'parameter setting interface', press numeric key "5" to turn to restore default interface, select "YES" or "NO".

Attention: After selecting 'Restore Default' the syringe parameter need re-calibration.

### **8.3.19 Maintenance reminder**

This function is mainly to remind the user to maintain the machine on a regular basis, set the parameters of the maintenance interval, when the machine detects that the setting of maintenance time has reached, it will remind user to maintain.

### **8.3.20 About this machine**

This option is mainly to display the machine model, software burning version, release version and the machine's SN code.

### **8.3.21 User-define drug library**

This function provides user-defined drugs (up to 30 drugs can be user-defined ) for injecting drugs in the Drug library mode. User-define drug can enter the drug name (up to 10 letters) and set the drug infusion mode (three modes are optional, the dose mode, body weight mode, flow rate mode) etc.

## **8.4 Operation Precautions**

- Avoid direct sunlight, high temperature and high humidity.
- If the pump work on battery only, please check battery capacity before operation and make sure it has enough power. Otherwise, recharge the battery fully.
- Avoid using the syringe pump with problems, which may cause medical accidents and bring harm to patient's health and even life.
- Only well-trained professionals are permitted to set or adjust infusion parameters.
- The Syringe Pump should be placed within 1.2 meters above or below patient's heat
- The damaged front panel (mask) need to be replaced in time to prevent leakage.
- Syringe Pump works under conditions that exceed the prescribed range may influence infusion accuracy or even cause malfunction.
- The degree of viscosity and ratio of medical liquid may influence infusion accuracy.
- The Syringe Pump uses 'Boon' brand syringe for factory setting. If users use the other brands of syringe, please calibrate its accuracy on machine before use.

## **8.5 Contraindications**

No findings so far.

## 9. Malfunctions Analysis and Solutions

Problems	Causes	Solutions
Accuracy discrepancy	The Syringe edge did not install into the syringe edge fixed groove	Please install it correctly
	The syringe currently used does not match the default brand	Select the correct brand of syringe or self-defined syringe
	Certain parts of the machine may be defective	Contact the distributor or manufacturer for repair
Push handle cannot move freely	There are liquid on the screw	Wipe with a wet clean soft cloth

Besides the problems mentioned in 8.2, please contact the sales agent/manufacturer for repair.

## 10. Safety Invention and Troubleshooting

### 10.1 Safety Invention and precautions

(1) AC power: built-in double fuses. When short circuit or any other malfunction occurs, the fuse shall cut off circuit in advance.

(2) DC input: built-in fuse. When short circuit or any other malfunction occurs, the fuse shall cut off circuit in advance.

(3) Battery protection: The battery contains protective devices against excessive pressure, over heat or short circuit, etc. to avoid overheating or burnt.

### 10.2 Troubleshooting

(1) If the syringe Pump gives system error alarm, stop the operation and contact the sales agent for repair. It can be used again only after it is well repaired and tested. Syringe Pump working with malfunctions may incur unpredictable damage.

(2) If the syringe Pump caught fire or displays any other malfunction, please disconnect the power immediately and contact the sales agent /manufacturer.

## **11. Maintenance, Inspection, Repair and Recycling**

### **11.1 Routine maintenance**

Routine maintenance includes the cleaning of outer shell and pump body. Clean it with wet soft cloth. Do not use solvents like xylene or acetone or other similar solvents which may corrode the syringe Pump.

### **11.2 Maintenance during operation**

The maintenance during operation mainly concerns the cleaning push handle and surrounding areas. Medical liquid may drip into the Syringe Pump during infusion process. Certain medical fluid may corrode the pump body; therefore clean the Syringe Pump every time after infusion completion.

### **11.3 Periodic Inspection**

#### **11.3.1 Inspect infusion accuracy (once every 2 months)**

Inspect periodically, if it is inaccurate please contact the sales agent /manufacturer.

#### **11.3.2 Inspect Internal Battery**

The battery shall reduce the performance due to prolonged usage, please check the battery capacity every other month.

(1) First recharge the battery fully (10 hours with power on, or 3 hours with power off).

(2) Let Infusion Pump work on battery only and set flow rate at 5ml/h. Record the whole working time when the battery is exhausted.

--If infusion time more than 360 minutes, the battery is in good condition.

--If Infusion time more than 240 minutes but less than 360 minutes, the battery starts low quality but still can be used.

--If infusion time less than 180 minutes, the battery reaches the end of its life and needs to be replaced.

Replace internal battery

(1) Unscrew the screws at the bottom of machine; remove the battery cover.

(2) Unplug the battery cable and take out the battery.

(3) Install the new battery. Please make sure the battery cable won't be squeezed by the battery Cover. Then install battery cover. After replacing new battery, please check its working condition.

## 11.4 Normal repair procedures

The repair job should be performed by supplier or distributor. It needs to make a complete inspection on machine after maintenance. If necessary, our company can offer circuit diagram and components list to authorized maintenance personnel.

## 11.5 Maintenance for long-time storage

If the Syringe Pump will not be used for long time, it should be placed in packing carton and avoid direct sunlight and keep it in cool and dry place. Refer to 12.2 for detailed storage conditions.

When using an Syringe Pump of long time storage, please refer to following steps before use:

- (1) Calibrate the Syringe Pump to ensure infusion accuracy and avoid possible medical accident.
- (2) Test occlusion alarm.
- (3) Test the working time and recharging time of battery to ensure the battery can still be used.

## 11.6 Recycling

The machines and its cable which have been used over its life span should be scrapped. For more information, please contact manufacturer or our distributors. ( Whether it is used frequently or not and whether it is repaired properly or not will impact infusion pump's life span.)

- (1) The scrapped syringe pump can be sent back to manufacturer or distributor.
- (2) The used battery can be sent back to manufacturer or distributor, or can be scrapped according to legally proper way.

## 12. Transport and storage

### 12.1 Precautions during transport

- (1) Place the product as per No. of layers indicated on packing carton.
- (2) Temperature:  $-20^{\circ}\text{C} \sim 60^{\circ}\text{C}$ ;
- (3) Relative humidity: 10~95% (no frosting)
- (4) Atmosphere pressure: 50.0kPa~106.0kPa

### 12.2 Storage conditions.

Storage temperature:  $-20^{\circ}\text{C} \sim +45^{\circ}\text{C}$  (With battery);  $-20^{\circ}\text{C} \sim +60^{\circ}\text{C}$  (Without battery)

Relative humidity: 10~95% (no frosting)

Atmosphere pressure: 50.0kPa~106.0kPa

### 13. Package list

Standard configuration in a package:

① Syringe pump	1 unit
② AC power cord	1 set
③ User Manual	1 pc
④ Product qualification certificate	1 pc
⑤ Warranty card	1 pc

### 14. Open-package Inspection

Cautions for Open-package inspection:

- (1) Opening the packing carton carefully to avoid damaging the machine or its accessories.
- (2) Handle with care all items inside the package.
- (3) Keep all accessories, warranty card and User Manual well for future use and reference.
- (4) Keep some packing cartons in case of using them to deliver defective machines.
- (5) If there is any accessory lacking or damaged, please contact the supplier at the earliest.

### 15. After Sales Service

The warranty for the Infusion Pump is one (1) year.

Note: The following situation is not within the range of free maintenance and repair

- (1) Malfunctions resulting from improper operation, or modification / repair of the Infusion Pump without supplier's knowledge and permission
- (2) Bruise or damage caused by improper handling during transport.
- (3) Malfunction or damage caused by fire, salt, poisonous gas, earthquake, hurricane, flood, abnormal electric voltage or any other natural disaster.

For all the malfunctions and damage due to above reasons, the manufacturer can offer repair but charge for the cost.

## Annex

**Table 1 Classification of alarms and color of alarm indicator light**


Classification of alarms	Alarm priority	Color and frequency of alarm indicator light
Handle off alarm	High priority	Red/2Hz
Syringe off alarm	High priority	Red/2Hz
Occlusion alarm	High priority	Red/2Hz
Low battery alarm	High priority	Red/2Hz
B.Exhaust alarm	High priority	Red/2Hz
Finished alarm	High priority	Red/2Hz
Empty	High priority	Red / 2Hz
Near Empty	Middle priority	Yellow / 0.5Hz
Almost Done alarm	Low priority	Yellow, steady
Use Battery alarm	Low priority	Yellow, steady
AC Fail alarm	Low priority	Yellow,steady
No Operation alarm	Low priority	Yellow,steady

**Table 2 Alarm conditions and alarm signal delay**

Names of alarm	Alarm condition delay	Alarm signal delay
Handle off alarm	10ms	100ms
Syringe off alarm	10ms	100ms
Occlusion alarm	840s@1ml/h 27s@25ml/h	100ms
LowBattery alarm	10ms	100ms
B.Exhaust alarm	500ms	100ms
empty alarm	10ms	100ms
Almost empty	10ms	100ms
AlmostDone alarm	10ms	200ms
Finished alarm	10ms	200ms
Use battery alarm	10ms	200ms

AC fail alarm	10ms	200ms
No Operation alarm	120ms	200ms

**Table 3 Characteristic parameters of alarm signals**

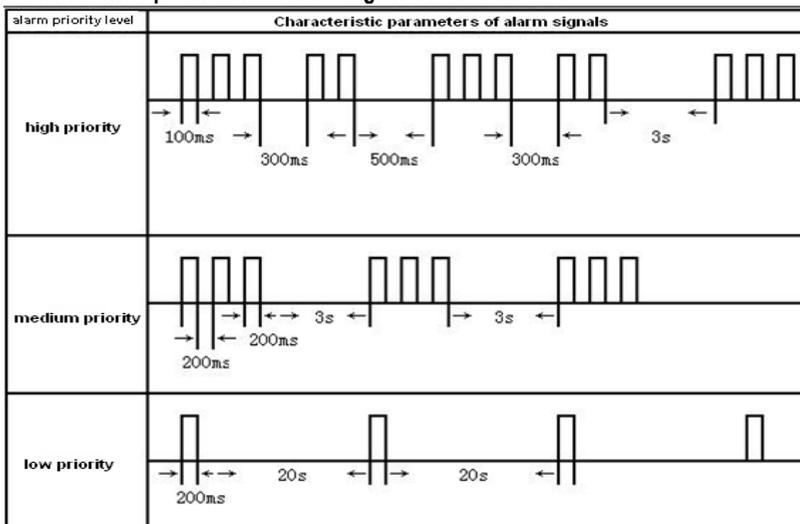

**Table 4 Occlusion response characteristic**


Flow Rate(ml/h)	OCCI alarm level	Occlusion pressure (KPa)	Occlusion alarm time
1	Low	41	12min59sec47s
	Middle	104	35min29sec
	High	144	42min53sec
5	Low	54	4min37sec69s
	Middle	90	7min09sec
	High	153	9min50sec

The above test uses 'Boon' brand of 5ml syringe. All the data are obtained by using 'Boon' brand syringe.


The syringe pump has pressure release function. When occlusion alarm sounds, the pressure in the

infusion line system will release automatically, so the bolus volume could be neglected when occlusion block release.

**Table 5 Starting Curves**


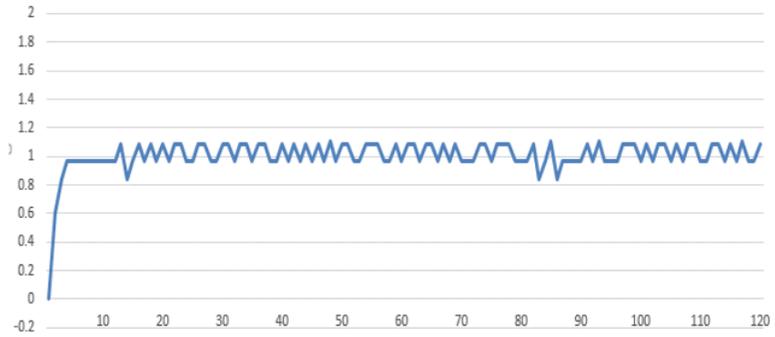


1ml starting curve


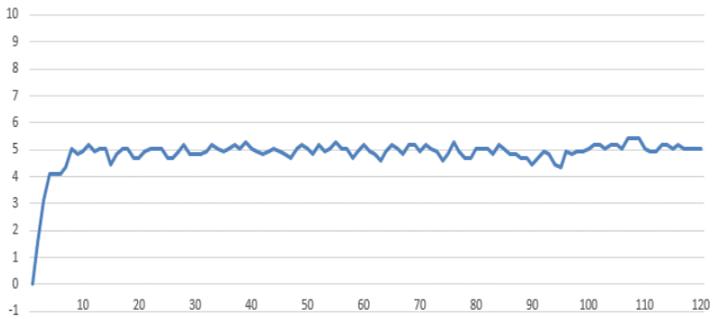


5ml starting curve

**Table 6 Trumpet Curves**

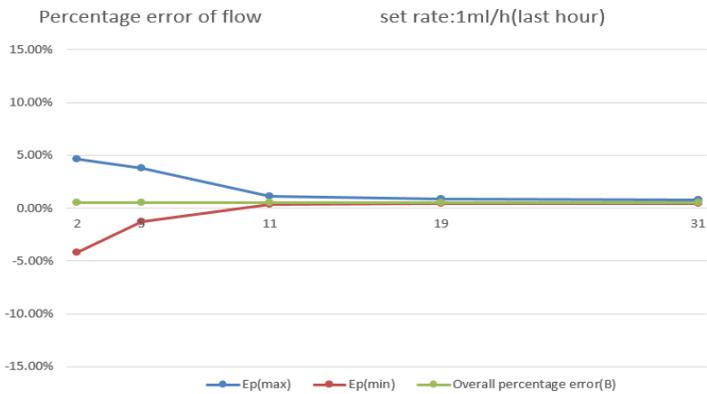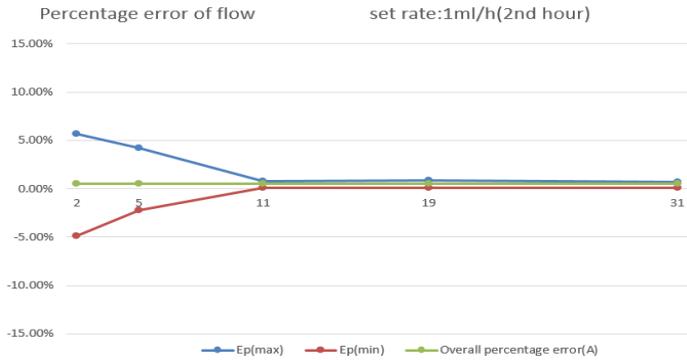

1ml Trumpet Curves

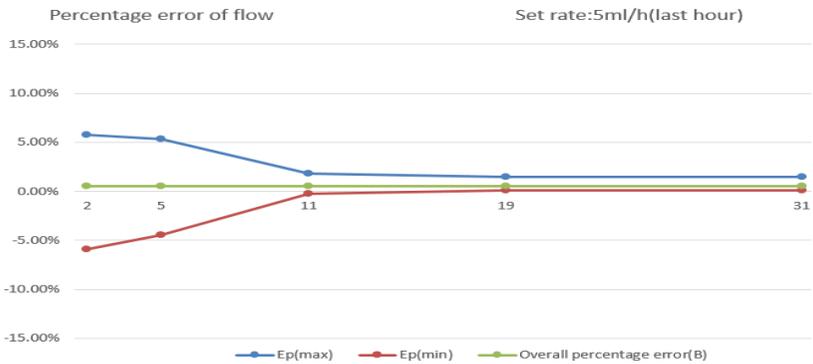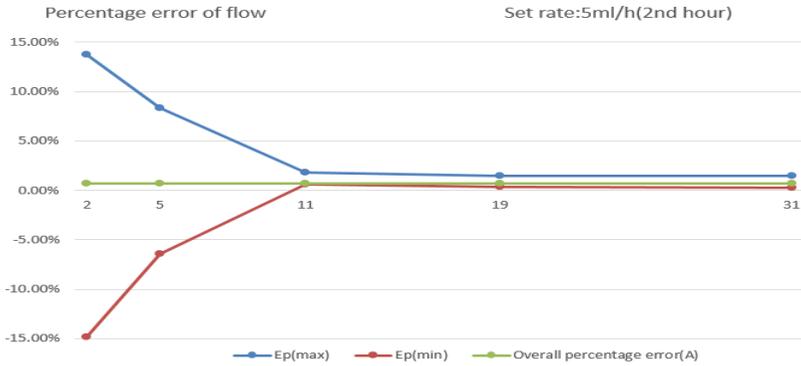

### 5ml Trumpet Curves

These data are testing result according to GB9706.27-2005 and the company's products standard. It use syringe pump and 10ml syringe under Boon brand.