

Alcohol-free Surface Disinfectant Wipes

30sec

2in1

Mint

x200

! Always Read Product Label and Material Safety Data Sheet before use. For Professional use only.

- Specially formulated and effective against all enveloped viruses, including Coronavirus, HIV, Hepatitis, Influenza and many other microorganisms. (for full details, see overleaf)
- For the disinfection of veterinary surfaces.
- Ideal to use on surfaces sensitive to alcohol.
- Wipes have a peppermint aroma and are supplied in dispenser tubs of 200 wipes. Tubs can be reused and refilled with G9 wipes refill packs.
- Each wipe measures 200 x 200mm.
- 2 in 1 product, wipes contain detergent for added cleaning power.

g9chemicals.com



Your only solution

Alcohol-free Surface Disinfectant Wipes

30sec

2in1

Mint

x200

! Always Read Product Label and Material Safety Data Sheet before use. For Professional use only.

Contact Times

Bactericidal – 30 sec
Fungicidal – 30 sec
Yeasticidal – 30 sec
Enveloped Viruses – 2 min
Mycobactericidal – 2 min
Spores – 5 min

Disinfectant Test Portfolio

The following test standards have been achieved: EN14885, EN14476, EN14348, EN13727, EN13704, EN13624

Bactericidal:

Pseudomonas aeruginosa,
Enterococcus hirae,
Staphylococcus aureus,
MRSA (60 sec), VRE (60 sec)

Mycobactericidal:

Mycobacterium avium

Yeasticidal:

Candida albicans

Enveloped Viruses:

Poxviridae, Herpesviridae,
Filoviridae (e.g. Ebola, Marburg),
Flavivirus, Hepatitis B virus (HBV),
Hepatitis C virus (HCV), Hepatitis
Delta virus (HDV), Influenza virus,
Paramyxoviridae, Rubella virus,
Measles virus, Rabies virus,
Coronavirus (e.g. SARS, MERS),
Human immunodeficiency virus
(HIV) & Human T cell leukemia virus
(HTLV)

Fungicidal:

Aspergillus niger

Spores:

Clostridium difficile

Directions For Use:

Using a new wipe for each application, wipe objects to be disinfected so that they are thoroughly moistened and let the disinfectant take action over a minimum of 60 seconds (to achieve full efficacy, leave for five minutes). Allow surface to dry or wipe with clean tissue.

g9chemicals.com



G9 Chemicals Ltd, 24 High St,
Charing, Ashford, Kent,
TN27 0HX, UK

+44 (0)1580 895 141
info@g9chemicals.co.uk
g9chemicals.com