

AEOLUS


Professional Vet Equipments Supplier




AEOLUS

X-style Electric Lifting Examination Table

FT-861 Series

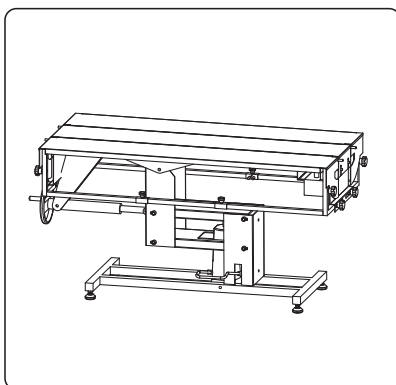

FT-826

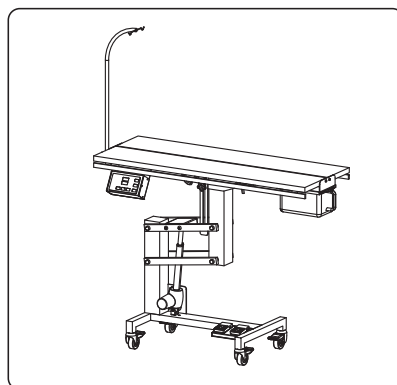

FT-828H

Distributor Contacts



DALLAS

11307 Indian Trail, Dallas, Texas 75229, USA.
Tel: +1 469 930 6359

SHANGHAI

Floor 5, Building 6, No. 675, Zhujin Road, Songjiang District, Shanghai, China.
Tel: +86-21-67632060, 67632364



New
Generation

User Manual

Package Inspection

The manufacturer is not liable for any damage caused by a third party "Transport or Delivery Company". The day the unit is delivered, an inspection of the outer packing should be made for any damage to the packing box.

Then the unit should be unpacked carefully; all components, accessories and the table itself should be inspected for any damage or loss.

If any damage is evident, the carrier or dealer should be notified immediately.

Brief Introduction

SUS304 stainless steel table top and electric lifting base provide a great table you'll rely on for years.

Lower down to 31cm (12 1/4"), enabling older, infirm and larger dogs to easily walk on.

World top class linear actuator provides outstanding loading capacity of 150kg (300lbs), powered by Linak.

Innovative X-style frame is stable, powerful and quiet.

The whole table may be equipped with casters or adjustable leveler.

Drainage hole (for sunken tabletop) at corner makes waste liquid flow away easily.

Incredibly stable X shape lifting base, discourages wobbling at any height.

Equipped with a height adjustable IV pole.

The FT-861 without the affix of -ECO will be equipped with a sunken tabletop and a stainless steel base.

The FT-861 affixed with -ECO will be equipped with a powder-coated base.

The FT-861 affixed with -BTR will be equipped with battery for emergency power supply.

The FT-861 affixed with -FLT will be equipped with a flat tabletop without drainage hole.

The FT-861 affixed with -SC will be equipped with a scale (flat top) instead of a tabletop.


The FT-861 affixed with -TLT will be equipped a tilting sunken tabletop.

Parts List of FT-861

Main part of Lifting table (tabletop included)-----	Quantity:1
IV pole-----	Quantity:1
Height-adjusting and tightening knob for IV pole -----	Quantity:1
Casters/Adjustable leveler(3 rubber and 1 grounded) -----	Quantity:4
Power cord-----	Quantity:1

Overview

- 1.Stainless steel tabletop (Sunken one with a drain hole or Flat without drain holes)
- 2.IV pole
- 3.Height-adjusting and tightening knob for IV pole clamp
- 4.Pet restraint
- 5.Foot pedal
- 6.Casters/Adjustable leveler
- 7.Linear actuator
8. Battery (for models with the affix of -BTR)

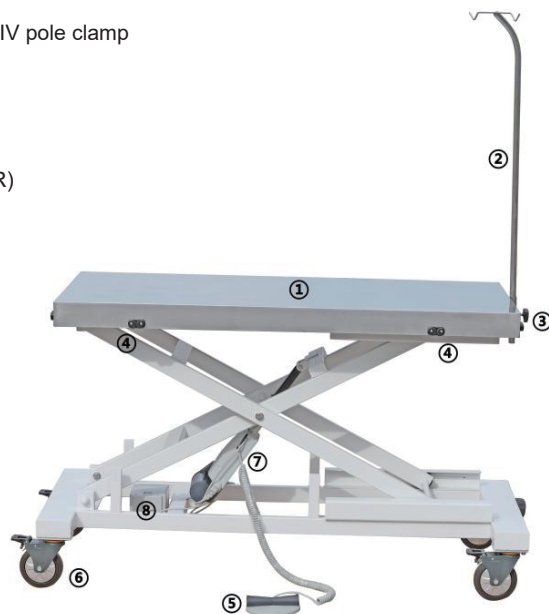

Parameters

Product dimension: 48 1/32"L×24 1/64"W×12 19/32 -41 9/64"H
122cmL×61cmW×32cm-104.5cmH
Package dimension: 53 5/32"L× 27 9/16"×17 21/64"H
135cmL×70cmW×44cmH
Net weight: 60kg
Gross weight: 90kg
Loading ability: 120kg

The following parameters are for models affixed with -BTR

Battery type: Lead acid (VRLA)
Capacity of battery: 1.3Ah 24V
Charging time: 10 hours
Continuous working time of the linear actuator supplied with the battery: ≤15 minutes

Safety Warning

1. Use only as described in this manual. Use only manufacturer's recommended attachments.
2. To reduce the risk of fire, or electric shock – DO NOT use outdoors or on wet surfaces. DO NOT place the product on an uneven or soft ground.
3. This product is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the product by a person responsible for their safety. Children should be supervised to ensure that they do not play with the product.
4. DO NOT use with damaged cord or plug.
5. DO NOT cut, damage, or modify the cord. DO NOT pull or carry by cord, use cord as a handle, close door on cord, or pull cord around sharp edges or corners. DO NOT pull, twist, or bend the cord with unnecessary force. DO NOT run the product over cord. Keep cord away from heated surfaces. DO NOT place heavy objects on the cord or allow the cord to get pinched in between objects. DO NOT wind the cord around the product.
6. DO NOT unplug by pulling on cord. To unplug, grasp the plug, not the cord.
7. DO NOT handle plug or the product with wet hands.
8. Turn off all controls before unplugging.
9. DO NOT use the product near flammable or combustible liquids such as gasoline or use in areas where they may be present. The fumes from these substances can create a fire hazard or explosion.
10. DO NOT modify, disassemble or repair.
11. Make sure the local voltage and frequency of the power meet the requirements on the label of the product.
12. Avoid using the product in dusty, vibrating and wet environment and in chemical storage place.
13. Neither people nor animals should be under the tabletop when it is power-on.
14. Gently control the tub to rise or lower slowly and safely.
15. Press the button in the air switch at least once a month.

Assembly Instructions

Casters/Levelers Installation

Screw the casters or the levelers into the screw holes at the bottom of the base.

IV pole installation

Put the IV pole through the hole at one end of the tabletop and rotate the tightener to fix the pole.

Charging for Models affixed with -BTR

When the battery is running low, the buzzer will send alarming sound to remind user to charge the battery. Connect the plug to the AC power supply, the indicator on the battery will be on and when the battery is fully charged, the battery light will be off.



Figure 1

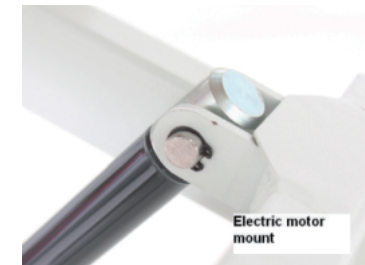


Figure 2

Operation Instructions

1. Plug in.
3. Step on the +(up)/-(down) foot pedals to adjust the height of the table as needed.
4. Install the IV pole inside the installation hole.
5. Put the animal on the tabletop and start working.
6. After work is completed, remove the animal, unplug the table, organize your tools and clean the tabletop.
7. For the table with a tilting tabletop, users can tilt the tabletop to drain the waste liquid on the tabletop.

Parts Replacement

Electric motor replacement

1. Disconnect the power cord from the wall socket. Remove the tabletop from the frame assembly, and set it aside. Remove the grooming arm.
2. Disconnect the electric motor cable from the motor control unit. Remove the snap rings and the retaining bolts that hold the electric motor to the frame assembly. (See Figure 1 and 2)
3. Using help, lift the frame assembly up, and remove the entire motor assembly from the frame.
4. Install the new motor assembly in the reverse order. Reconnect the cable to the motor control unit.
5. Plug the power cord into a wall socket and test the table for proper operation.

Service Support

Contact us if you cannot solve your problem with this manual.

Dallas

11307 Indian Trail, Dallas, Texas 75229, USA.

Tel: +1 469 930 6359

Web: www.aeoluspet.com

Shanghai

Floor 5, Building 6, No. 675, Zhujin Road, Songjiang District, Shanghai, China.

Tel: +86-21-67632060, 67632364