

## MOECK WARMING MATS - INSTRUCTIONS


Here are the most important points that will be included in the new washing instructions. Perhaps you can pass these on to your customers now:

1. Correct folding of the blanket: The blanket must be folded with the green side facing outwards and may only be folded lengthways once.
2. Spin speed: Please ensure that the spin speed does not exceed 800 revolutions per minute.
3. Avoid eco-washing programs: Avoid using eco-washing programs, as these programs do not pump enough water into the washing machine.

Enclosed you will find photos that illustrate the correct application.

***Please find images below on how to fold the mats correctly:***

Step 1

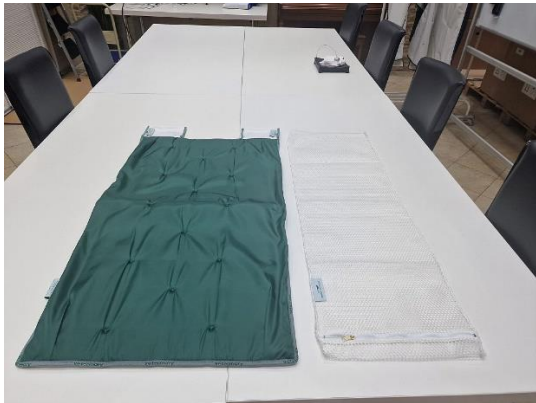

Step 2



Step 3



Step 4

