



Cleaning/Disinfection Guide

Aesculap Sterile Containers



Regular maintenance and proper care of your sterile containers is essential for ensuring its longevity and optimal performance. By following these simple steps, you can keep your sterile containers in excellent condition whilst in use in practice.

A) Manual Cleaning / Disinfecting

- 1 Prior to manual disinfecting, allow water to drip off for a sufficient length of time to prevent dilution of the disinfecting solution
- 2 After manual cleaning/disinfection, check visible surfaces visually for residues. Repeat the cleaning process if necessary

Manual cleaning with disinfecting cleaning by wipe disinfection

D-W: Drinking Water FD-W: Demineralized Water RT: Room Temperature

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water Quality	Chemicals	
I	Cleaning	RT (cold)	-	-	D-W	-	-
II	Dry	RT	-	-	-	-	-
III	Wipe disinfection	-	>1	-	-	A Alcohol Denat. 70% (Ethanol B. Braun)	B Aldehyde-free surface disinfectants (e.g. <u>G9 Alcohol Wipes</u>)
IV	Final rinse	RT (cold)	0.5	-	FD-W	-	Rinse off cleaning chemicals; do not leave residues
V	Dry	RT	-	-	-	-	-



Manual Cleaning/Disinfecting Cont.

- Phase I** Clean the **product** under running tap water using a suitable cleaning brush until all visible residue has been removed from the surfaces.
Move non-rigid components, such as adjusting screws, joints etc. during cleaning.
- Phase II** During the **drying phase**, dry the product using the appropriate equipment (e.g. wipes, compressed air).
- Phase III** Wipe all surfaces of the product using a **single-use disinfecting wipe**.
- Phase IV** **Rinse** disinfected surfaces after the prescribed contact time (at least 1 min) under running demineralized water.
Drain any water fully.
- Phase V** During the **drying phase**, dry the product using the appropriate equipment (e.g. wipes, compressed air).

B) Mechanical Cleaning/Disinfecting

Mechanical neutral cleaning and thermal Disinfecting

Machine type: single-chamber cleaning/disinfection without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water Quality	Chemicals
I	Pre-rinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	Neutral cleaner (e.g. B. Braun Helimatic Cleaner neutral in 0.5% working solution): pH neutral (pH 6.5 to 8.5)
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disinfection	90/194	5	FD-W	Other process parameters may be feasible
V	Dry	120/248	10	-	

D-W: Drinking Water

FD-W: Fully desalinated, demineralized water (low-germ, max 10CFU / 100ml, as well as low endotoxin contamination, max 0.25 endotoxin units/ml)



Mechanical Cleaning/Disinfecting Cont.

- 1 Check visible surfaces for residues after mechanical cleaning/disinfecting
- 2 If white residue is observed on the container, this may have been caused by a high pH (e.g. caused by alkaline cleaning solution), water quality or deviating process parameters.
- 3 Check pH level of water and detergent solution throughout the process – reduce to a pH of 6.5 to 8.5.
- 4 The white residue does not impact form, fit or function
- 5 Repeat the cleaning/disinfection process if necessary

Note: Temperatures of up to 120°C are permitted for machine drying with hot air

Care of your sterilisation containers

Oil closure hinges with suitable maintenance oil
e.g. Aesculap Sterilit i Lubricant Spray (SLSPRAY), or Maintenance Oil (SL50).

Recommended Products



G9200AW
G9 Alcohol
Disinfectant Wipes
- Large Drum



G9200AF
G9 Alcohol
Disinfectant Wipes
- Refill Pack

200
WIPES

For more
information
contact our
Technical Team on
0300 068 3300

or email
**tech@vet-
direct.com**

