



Pour Fill Vaporiser Filling & Draining Instructions

Pour (Screw Cap) Filler

-  **Warning:** Dial should be in 'OFF' position. Remove filler cap by turning cap counterclockwise. Be sure drain plug is closed.
-  **Caution:** Turn cap slowly if vaporiser is pressurised
-  **Warning:** Check that the drug is the same as specified on the vaporiser. Pour agent slowly into opening. Observe agent level through sight glass.



If the vaporiser is dry the level will fall slightly as the wicks absorb the agent. Replace cap by turning clockwise. The cap should be tight.

The Vaporiser is now ready for use

Liquid Capacity

Amount of Anaesthetic Agent to fully charge the vaporiser = 150ml (nominal)
Amount retained by Wick System = 35ml (nominal)
During initial fill, level may not begin to show until the wick is fully saturated

Draining

To Drain: remove the filler cap to reveal drain plug, using inverted filler cap unscrew the plug, but do not remove. Drain only into a properly marked container.

