


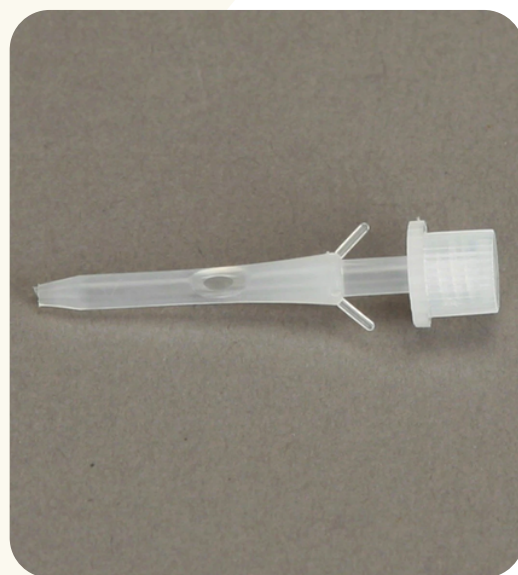

Instructions for Using an Aural Haematoma Tube

1. **Local Anaesthetic (optional, dependant on clinician preference):** Administer a local anaesthetic to the affected ear.
2. **Incision:** Make a small incision in the distal medial surface of the ear to allow for drainage.
3. **Drainage:** Allow the hematoma to drain naturally or by gently massaging the area.
4. **Tube Placement:** Introduce the aural hematoma tube into the incision, ensuring the flexible fingers of the tube hold it in place.
5. **Removal:** Sharp pull on the device should remove it with ease.



Benefits of Using Aural Haematoma Tubes

- **Minimally Invasive:** Avoids the need for general anaesthesia and extensive surgery.
- **Continuous Drainage:** Allows for continuous drainage, promoting healing and preventing reaccumulation of fluid.
- **Cost-Effective:** A less expensive alternative to traditional surgery.
- **Comfortable for the Animal:** The tube is designed to be comfortable for the animal during the healing process.

