




Key Fill Vaporiser Filling & Draining Instructions

Keyfiller – Filling

! **Warning:** Fill the vaporiser only when the control dial is in the OFF position and do not turn the dial on during filling or attempt to fill beyond the 'FULL' mark.

! **Caution:** Always hold the bottle below the level of the end of the filling adaptor and also below the level of the filler port until the adaptor is clamped in position.

! **Caution:** Turn screw slowly if vaporiser is pressurised. Turn top retaining screw on Vaporiser Filler unit counterclockwise and take out Filler Plug.

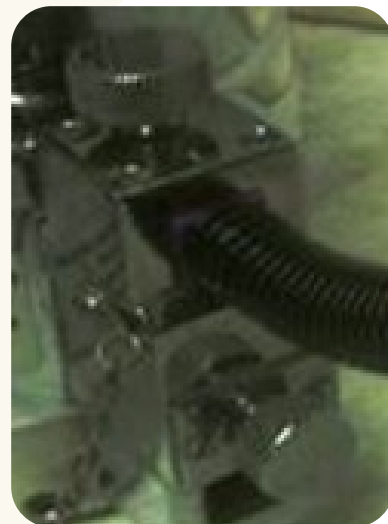

1. Insert the end of the Adaptor with the two holes downwards into the top filler port, take care to bend Tube down slightly so bottle is below inlet level.
2. After insertion, turn the retaining screw clockwise and tighten to seal Filler Adaptor in the Filler Socket.
3. Raise the bottle above the level of the filler port now and begin filling.
Note: The stream of bubbles inside the bottle will confirm that filling is taking place. The Vaporiser will fill in 20-30 secs. When the vaporiser is filled to the full level lower the Bottle and remove the Adaptor.
4. Replace the Filler Plug and tighten the Top retaining screw again.

The Vaporiser is now ready for use

Liquid Capacity

Amount of Anaesthetic Agent to fully charge the vaporiser = 150ml (nominal)
Amount retained by Wick System = 35ml (nominal)
During initial fill, level may not begin to show until the wick is fully saturated



Draining



1. Unscrew the bottom retaining screw and insert the Adaptor into the bottom socket and tighten again.
2. Ensure that the bottle is held below the level of the end of the Adaptor and open the Drain Valve.
Note: To let air vent from the bottle remove the filler plug.
3. When the vaporiser is drained remove Adaptor and tighten the Drain Valve and bottom retaining screw.

